BPA-Free

Sin BPA

Sans BPA

BPA-Free

This product is made from Polyethersulfone (PES) – a BPA-Free material with a natural honey-colored tint.

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FR Appelez SANS FRAIS : **1.800.54.AVENT**

www.philips.com/AVENT

USA: Philips Consumer Lifestyle, a division of Philips Electronics North America Corporation, P.O. Box 10313, Stamford, CT 06904.

Canada: Philips Electronics Ltd, 281 Hillmount Road, Markham, ON L6C2S3

Trademarks owned by the Philips Group.
© Koninklijke Philips Electronics N.V. All Rights Reserved.

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FR Tire lait Electronique Double

Twin Electronic Breast Pump Parts Guide

When assembled parts b – g make up an assembled pump body. When the Control handle (a) is added to this assembled pump body, it becomes the First (Control) Pump. When the Double pumping connector (l) is added to this assembled pump body, it becomes the Second Pump.

First (Control) Pump

Second Pump

Pump Motor

- a 1x Control handle
- b 2x Silicone diaphragm
- c 2x White valve
- d 2x Pump body
- e 2x Milk container (2x 4 oz/125 ml)
- f 2x Stand (same as h)
- g 2x Let-down Massage Cushion
- h 2x Funnel cover (same as f)
- i 1x Control cable
- j 2x Transparent tube
- k 1x Twin tubing
- l 1x Double pumping connector
- m 1x Pump motor
- n 1x Electric adaptor
- o 1x Wall plug cord
- p 2x Dome cap
- q 2x Extra Soft, Newborn Flow Nipple (0m+)
- r 2x Screw ring
- s 2x Sealing base
- t 2x Sealing disc

All you need to assemble and use the breast pump is shown here. If any of the parts are missing, or if you have any questions, please call customer support at 1.800.54.AVENT or visit www.philips.com/AVENT

Twin Electronic Breast Pump Parts Guide – Detailed

Control Handle

Pump Motor

IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed.

READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

WARNING – To reduce the risk of fire, electrocution, or other personal injury:

- The breast pump is intended for repeated use by a single user only.
- Always unplug the breast pump motor from the electrical outlet immediately after use.
- Do not use while bathing.
- Do not place or store the product where it can be pulled or fall into water.
- Do not place or drop the pump motor, control handle, electric adaptor or wall plug cord into water or other liquid.
- Do not reach for a product that has fallen into water. Unplug from electrical outlet immediately.
- Close supervision is necessary when this product is used near children or persons with disabilities.
- Do not allow children or pets to play with the pump motor, tubing, power supply or accessories as these parts may pose a choking or strangulation danger.
- There are no serviceable parts inside the breast pump. Do not attempt to open, service or repair the breast pump yourself.
- Before each use inspect the product, including mains cord, for signs of damage. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or has been submerged in water.
- Only use attachments recommended by Philips AVENT.
- Do not use outdoors with the AC adaptor, where aerosol spray products are being used, or where oxygen is being administered.
- Equipment not suitable for use in the presence of flammable anesthetic mixture with air, or with oxygen or nitrous oxide.
- Only use the electric adaptor supplied with the product.
- Ensure the unit is not exposed to extreme heat or placed in direct sunlight.

- This product is a personal care item and should not be shared between mothers.
- Do not attempt to remove the pump body from your breast while under vacuum. Turn the unit off and break the seal between your breast and the pump funnel with your finger, then remove the pump from your breast.
- Never use while pregnant, as pumping can induce labor.
- Never use when sleepy or drowsy.

CAUTION – To reduce the risks of injury, device malfunction or failure:

- Do not leave product unattended when plugged into an electrical outlet.
- Never drop or insert any object into any opening or tubing.
- Do not shorten the tubing.
- Make sure the voltage of the electrical adaptor is compatible with the power source.
- First plug the adaptor into the breast pump motor. Then, attach the wall plug cord into the adaptor. Lastly, insert plug into the wall socket.
- Although compliant with applicable EMC directives, this equipment may still be susceptible to excessive emissions and/or may interfere with other more sensitive material.
- Wash and sanitize the appropriate pump components before each use.
- Do not continue pumping for more than five minutes at a time if no result is achieved.
- Do not use antibacterial or abrasive cleaners/detergents when cleaning breast pump parts.
- Never put the control handle or breast pump motor in water or a sterilizer, as you can cause permanent damage to the pump.
- If the pressure generated is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the pump body with your finger and remove the pump from your breast.

SAVE THESE INSTRUCTIONS

Congratulations!

Soon you will discover what makes this Philips AVENT Twin Electronic Breast Pump so uniquely yours. The breast pump is designed with an electronic memory that learns and continues your gentle pumping rhythm. It gives you complete control over your comfort, which means that pumping with your Twin Electronic Breast Pump always feels natural and exactly right for you.

Let-down Massage Cushion has petals that flex in and out to gently massage the areola. This is intended to help stimulate a fast, natural milk flow.

One-Touch Memory Control is right at your fingertips. You can stop, start or adjust the speed and suction level without removing your hand from the pump.

Electronic Memory learns from you. Just begin a comfortable pumping rhythm, press a button and the Twin Electronic Breast Pump automates it.

Pump, Store, Feed Convenience from the Philips AVENT Natural Feeding System. Products work together, so you never have to transfer milk between containers.

Ultra Comfortable, Quiet and Effective making it easier for you to continue giving your baby the benefits of breast milk.

Please take a few minutes to read through this entire instruction leaflet before using your breast pump for the first time.

PLEASE SAVE THESE INSTRUCTIONS.

Unless advised otherwise by your health professional it is best to wait until your breastfeeding routine is established before expressing – usually around 4 weeks.

What's Inside:

Electric Pumping (Single and Double)	
Twin Electronic Breast Pump Parts Guide	1-2
Important Safeguards	3
Disassembling Your Electronic Pump	5
Cleaning and Sanitizing Your Pump	6
Preparing to Pump	7-8
Assembling the Pump Body	9-10
Connecting the Electronics	11-12
Pumping with Your Electronic Breast Pump	13-14
Keeping Parts Clean and Compact for Travel	
Storing Your Expressed Breast Milk	15
Troubleshooting Guide	16-17
Technical Specifications	18
Limited Warranty	19

Disassembling Your Electronic Pump

Make sure that your breast pump motor is turned off and unplugged before disassembling parts. See section titled, 'Twin Electronic Breast Pump Parts Guide' on page 1 for a description of the First (Control) Pump and Second Pump.

First (Control) Pump

First (Control) Pump

- 1 On the first (control) pump, remove the control handle by pulling it away from the silicone diaphragm.

Second Pump

Second Pump

- 2 On the second pump, detach the transparent tube from the double pumping connector. Remove the double pumping connector by pulling it away from the silicone diaphragm.

Perform steps 3-5 on both the First (Control) Pump and Second Pump.

- 3 Remove the silicone diaphragm by pulling it away from the pump body.

- 4 Remove the white valve from the pump body.

- 5 Remove the Let-down Massage Cushion by gently pulling it away from the funnel on the pump body.

Cleaning and Sanitizing Your Pump

Make sure that your breast pump is turned off and unplugged before cleaning.

Clean with a Damp Cloth when Necessary:

Control handle (a)

Tubing (i/j/k)

Note: The tubes are used for suction and do not carry milk.

Double pumping connector (l)

Pump motor (m)

Caution: NEVER put the control handle or pump motor in water or a sterilizer, as you can cause permanent damage to the pump.

Clean Before Every Use (including first use):

Silicone diaphragm (b)

White valve (c)

Pump body (d)

Milk container (e)

Let-down Massage Cushion (g)

Funnel cover/White stand (h/f)

Cleaning Instructions:

These pump parts should be sanitized before every use.

Disassemble the pump before cleaning. Wash in warm soapy water and rinse, or wash on the top rack dishwasher. Sanitize in a steam sterilizer (digital or microwave) or by boiling in water for 5 minutes.

Caution: Do not use antibacterial or abrasive cleaners/detergents. To clean the valve, rub gently between your fingers in warm soapy water – do not insert objects as this may cause damage.

HELPFUL TIP!

The Philips AVENT Digital and Microwave Steam Sterilizers are designed to hold the washable components of the Electronic Breast Pump and the entire Manual Breast Pump.

When using your Twin Electronic Breast Pump for the first time, it will have retained factory settings stored in the memory as a result of quality control testing. If the blue One-Touch Memory Control is pressed prior to squeezing the control handle upon first use the pump will operate at these settings, which may or may not be comfortable for you. We recommend that prior to engaging the automatic mode, you establish a comfortable rhythm manually, then press the blue One-Touch Memory Control to continue your preferred rhythm.

If you find that you are not expressing a lot of milk the first few times you pump, don't be discouraged. Learning to express milk, like breastfeeding, takes practice. Here are some tips that might make pumping easier:

1. Choose a time when you're not rushed and won't be interrupted.
2. Choose a comfortable chair with a table nearby, so you can set the pump motor within reach.
3. Look at a photograph of your baby to help encourage let-down.
4. Place a warm cloth on your breasts for a few minutes before you start pumping. This encourages milk flow and can soothe painful breasts.
5. Warmth and relaxation can encourage milk flow. Try expressing after a bath or shower.
6. Try expressing from one breast while your baby is feeding from the other, or continue expressing just after a feed.
7. Reposition the pump on your breasts from time to time. This can stimulate your milk ducts.

How quickly will the milk flow begin?

When pumping with the Twin Electronic Breast Pump your milk should begin to flow after a few compressions. If your milk does not begin to flow immediately, relax and continue pumping. **CAUTION: DO NOT CONTINUE PUMPING FOR MORE THAN 5 MINUTES AT A TIME IF NO RESULT IS ACHIEVED.** Stop, and try expressing milk at another time of day. If the process becomes painful, stop and consult your health professional or breastfeeding advisor.

How much milk can you expect?

It has been shown that double pumping boosts prolactin levels and stimulates milk production so, if you can, use your breast pump regularly as a double pump. On average you will need to pump for 10 minutes to express 60-120 mls or 2-4 oz of milk. However, this is just a guide and varies from woman to woman. If you find that you regularly express more than 4 oz/125 ml of milk per session, attach a 9 oz/260 ml AVENT Breast Milk Container to the pump body.

You can use Twin Electronic Breast Pump as a:

Twin Electronic Pump

Single Electronic Pump

Assembling the Pump Body

Before assembling, make sure that you have correctly washed and sanitized the pump. See section titled 'Cleaning and Sanitizing Your Pump' on page 6.

Parts you will need for this step:

Pump body

White valve

Clear silicone diaphragm

Double pumping? Assemble two pumps as shown below.
Single pumping? Assemble one pump as shown below.

- 1 Insert the white valve into the pump body, so that it fits securely into the small round hole. Make sure the pointed side of the valve is facing down.

- 2 Fit the clear silicone diaphragm into the pump body. Ensure a perfect seal around the rim by pressing down with your fingers. (This is easier if assembled while the diaphragm is wet.)

Assembling the Pump Body

Let-down Massage Cushion

Milk container

White stand

- 3 Screw the pump body onto the milk container until securely in place.


To ensure that the pump is completely stable, you can use the stand with the 4 oz/125 ml milk container.

- 4 The Let-down Massage Cushion has petals that flex in and out to gently massage the area around the breast. This is intended to help stimulate a fast, natural milk flow. When using the breast pump without the Let-down Massage Cushion, the maximum vacuum level achievable will be higher.


Parts you will need for this step:

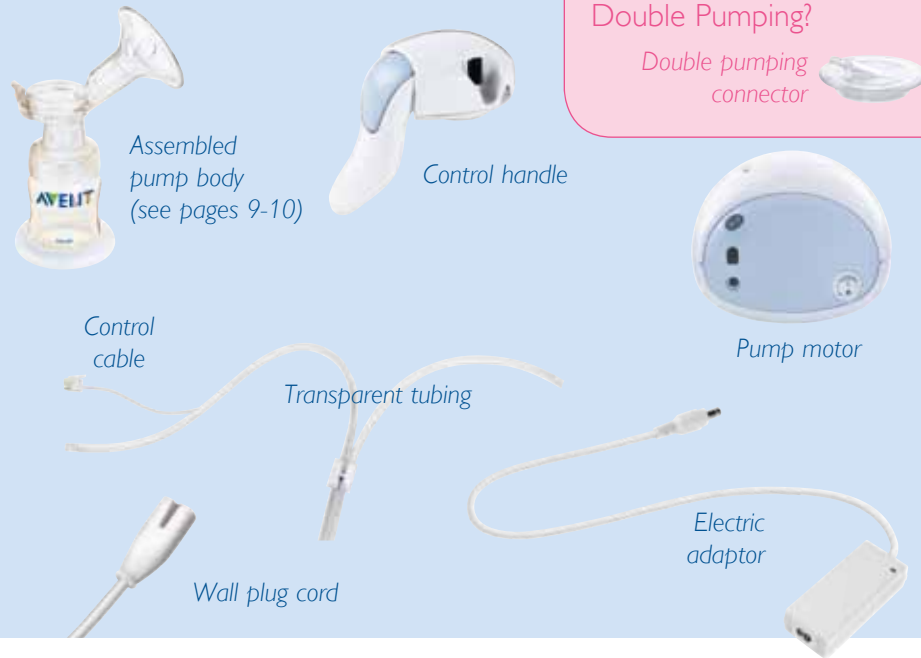First (Control) Pump

First (Control) Pump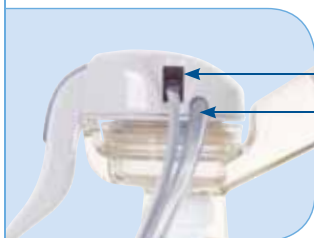

See section titled, 'Twin Electronic Breast Pump Parts Guide' on page 1 for a description of First (Control) Pump and Second Pump.


- 1 Push the control handle onto the assembled pump body so that it sits firmly on top of the silicone diaphragm. This assembly becomes the First (Control) Pump which contains the electronics of the pump.

- 2 Click the control cable into the socket on the control handle. Then securely attach one of the transparent tubes to the port on the control handle.

Double pumping? Complete the next step with your second assembled pump body.


See section titled, 'Twin Electronic Breast Pump Parts Guide' on page 1 for a description of First (Control) Pump and Second Pump

- 3a Push the double pumping connector onto the pump body so that it sits firmly on top of the silicone diaphragm. Securely attach the other transparent tube to the double pumping connector. When the breast pump is fully assembled, the Second Pump will mimic simultaneously the speed and suction of the First (Control) Pump.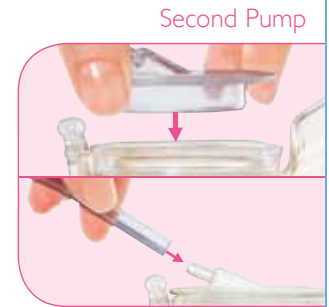

Single pumping? Complete the next step with the remaining tube.

- 3b If you are single pumping, you can keep the transparent tube out of the way by attaching it to the white clip.

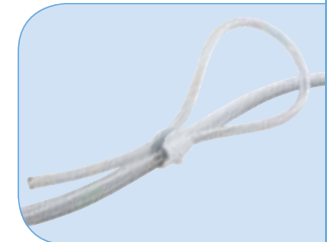- 4 Securely attach the twin tubing to the ports on the pump motor. Connect the control cable into the adjacent socket.

- 5 Connect the electric adaptor to power socket on the pump motor. Then connect the wall plug cord to the electric adaptor and insert the plug into the wall outlet.

Pumping with Your Electronic Breast Pump

Before you begin pumping, make sure that you have correctly washed, sanitized and assembled the pump. Consult the 'Preparing to Pump' Section on page 7 for tips on stimulating let-down and milk flow.

- 1 Wash your hands thoroughly and make sure your breasts are clean.
Turn on Pump: When the flashing **ORANGE** light changes to a continuous **GREEN** light, the pump is ready to use.
If the light is flashing **RED**, refer to the 'Troubleshooting Guide' on page 18.

- 2 Form a Seal: Press the pump against your breast, so that your nipple fits into the funnel. A seal should form between the pump and your breast. (If double pumping, press a pump to each breast.)

- 3 Let-down: Let-down is your body's physiological response to the suckling action of your baby on the breast. This stimulates the breast to produce milk and send it towards the nipple where it is available for baby.

To initiate let-down, begin by squeezing the handle with shallow quick strokes. This is intended to imitate your baby's natural suckling pattern. At the same time you are doing this, you are enabling the petals on the Let-down Massage Cushion to flex in and out around the areola, creating a massaging effect. This is intended to help stimulate the natural let-down reflex. Let-down can occur immediately after a few strokes and should not take longer than 5 minutes (refer to page 7 on 'Preparing to Pump'). You can choose to automate this let-down phase by pressing the blue One-Touch Memory Control or you can continue to pump manually.

- 4 Milk Expression: Milk Expression occurs after let-down is achieved and milk begins to flow. It is the phase that is induced by the baby's deeper more comfortable suckling rhythm.

If you did not automate the let-down rhythm, please skip to Step 5. If you have automated the let-down phase in Step 3, press the blue One-Touch Memory Control to end the let-down rhythm. You can now begin to establish a comfortable pumping rhythm and level of suction that can be maintained throughout your pumping session.

Pumping with Your Electronic Breast Pump

- 5 Your Electronic Breast Pump Mimics Your Pumping: To create a natural breastfeeding rhythm, we suggest you hold down on the handle for 3 seconds, then release and repeat the pattern for the next compression. **Next, press the blue One-Touch Memory Control on the handle.** The mode light on the pump motor will change from solid **GREEN** to flashing **GREEN** and your breast pump will mimic the rhythm of the last two compressions.

Note: You do not need to compress the handle fully, only until it feels comfortable. The further you compress the handle, the more suction the pump will provide. If the pressure is uncomfortable, stop the automated pumping by pressing the One-Touch Memory Control and breaking the seal between your breast and pump funnel with your finger.

- 6 Changing the Speed and Level of Suction: Press the blue One-Touch Memory Control on the handle again, and the pump will revert to manual control. The light will turn from flashing **GREEN** to solid **GREEN**. Establish a comfortable speed and level of suction. Press the blue One-Touch Memory Control again, and your Electronic Breast Pump will continue the same pumping rhythm.

- 7 Ending Pumping Session: Press the blue One-Touch Memory Control, break the seal between your breast and pump funnel with your finger and remove the pump from your breast. Turn the pump off and unplug at the power source. Your Electronic Breast Pump will remember and save the speed and suction level of your last two compressions. Unscrew the base after milk is collected. If any milk remains in the pump body, pour it into the base. Do not overfill the base. Wash and re-sanitize all necessary parts before the next use.

- 8 Next time you turn your pump on: *Resuming Saved Session?* Turn on the pump as in Step 1. Complete Step 2. Press the blue One-Touch Memory Control. This engages the automatic pumping mode. The pump will continue with the rhythm of the last two saved compressions of the milk expression phase. Continue pumping as described in Step 4. *Need to re-establish let-down?* If you need to establish let-down, turn on the pump as in Step 1. Complete Step 2. Squeeze the handle with shallow quick strokes and proceed as described in Step 3.

CONSULT THE GUIDELINES FOR STORING EXPRESSED BREAST MILK ON PAGE 16 & 17.

HELPFUL TIP!

The light on top of the Twin Electronic Breast Pump Motor indicates the pump mode:

- Flashing **ORANGE** Light: Pump is in warm-up mode. Wait for the continuous **GREEN** light.
- Continuous **GREEN** Light: Manual pumping mode. You control the speed and pressure of the pump.
- Flashing **GREEN** Light: Automatic pumping mode. Your breast pump will mimic your last established pumping rhythm and level of suction.
- Flashing **RED** Light: Consult the 'Troubleshooting Guide' on page 18.

Need Help? Call us at 1.800.54.AVENT or visit www.philips.com/AVENT

Pump

After sanitizing all parts insert the white valve and fit the diaphragm onto the pump body. Clip the funnel cover over the Let-down Massage Cushion and screw the sealing base onto the bottom of the pump body. Then put the assembled items in the gray drawstring bag.

Sealing base (s)

Pump body (d)

Let-down Massage Cushion (g)

Funnel cover (h) (also used as a stand when pumping)

Breast Milk Container

After sanitizing all parts, click the sealing disc into the screw ring and twist onto the container.

Screw ring (r)

Sealing disc (t)

Milk container (e)

AVENT Naturally Shaped Nipple

After sanitizing all parts, insert the nipple into the screw ring and attach the dome cap over the top of the nipple. Screw on the sealing base.

Dome cap (p)

Screw ring (r)

Nipple (q)

Sealing base (s)

Using Philips AVENT products interchangeably, you can pump, store and feed with the same container, preserving valuable nutrients that can get lost if breast milk is transferred.

After expressing milk into an AVENT container, click a sealing disc into the screw ring and twist onto the container. To feed, just replace the sealing disc with an AVENT Naturally Shaped Nipple.

Philips AVENT Breast Milk Storage System:

With your Twin Electronic Breast Pump or Manual Breast Pump, you can express milk directly into pre-sterilized Breast Milk Storage System Cups. These cups are safe, sturdy and offer protection for pumping, storing and feeding breast milk.

Get Philips AVENT Breast Milk Storage System and Get Going!

- Twist-on lid seals tightly to cup for leak-proof storing or freezing
- Writable and stackable cups make it easy to organize
- Compatible system lets you pump, store and feed using the same cup
- Sturdy cups heat easily and quickly in a bottle warmer

Guidelines for Storing Expressed Breast Milk

- Refrigerate breast milk immediately. Refrigerate for up to 24 hours in the back of the refrigerator, not in the door.
- Freeze for up to 3 months, or 6 months in a zero-degree freezer.
- Frozen breast milk can be defrosted overnight in the refrigerator, thawed in warm water or heated on the lowest setting of a bottle warmer or baby food warmer.
- NEVER refreeze breast milk.
- If you express small amounts of milk throughout the day, you can top up the milk container by keeping it in the refrigerator and adding to it. Make sure the milk is used within 24 hours of the first expression, or store it in the freezer.
- DO NOT add fresh breast milk to already frozen milk.
- ALWAYS discard any milk that is left over at the end of a feed.

Note: Only milk collected with a clean, sanitized pump should be stored to be fed to your baby. Sanitize Breast Milk Containers before use or express milk directly into pre-sterilized cups.

Never immerse a frozen bottle or liner in boiling water as this may cause it to crack/split.

Transporting Expressed Breast Milk

- Please refer to Tote Bag Instructions for complete directions on how to store and transport breast milk using the Thinsulate™ carriers.
- When removed from the freezer, four cool packs will keep one 4 oz /125 ml or one 9 oz /260 ml container of freshly expressed breast milk sufficiently chilled inside a Thinsulate carrier and insulated bag for up to 6 hours. Ensure that both the storage containers and carriers are placed upright in the bag to prevent leaking. Also make sure that the Thinsulate carriers are completely zipped.
- Be sure to keep the cool packs frozen until they are ready to use and refrigerate milk immediately after removing it from the insulated bags.
- In hot climates, avoid storing Thinsulate carriers or the travel bag containing expressed milk in a car, office, cabinet etc. Milk must be sufficiently chilled to avoid spoilage.

Please note: Milk storage times vary based on collection and storage conditions. This storage information is only a suggestion. Please contact your health professional or breastfeeding advisor about milk storage guidelines.

<i>Lack of suction</i>	◦ Check that all the pump components have been assembled correctly and that the white valve is positioned correctly. ◦ Ensure that the silicone diaphragm and Let-down Massage Cushion are fitted securely to the pump body and a perfect seal is created, making sure to push the Let-down Massage Cushion far enough down the center of the funnel. ◦ Make certain that the tubing connections and control handle are firmly in place.
<i>No milk being expressed</i>	◦ Ensure the pump is correctly assembled and that suction is being created. ◦ Relax and try again, as expression may improve after practice. ◦ Refer to the 'Preparing to Pump' section on page 7.
<i>Pain in the breast area when expressing</i>	◦ You may be pumping too hard. You do not need to use all the suction the pump can generate. Try compressing the handle halfway. ◦ Try holding down the pump handle for 2-3 seconds per compression. ◦ Consult your health professional or breastfeeding advisor.
<i>Milk is drawn up the Let-down Massage Cushion</i>	◦ Try leaning slightly forward. ◦ Remove and re-fit the Let-down Massage Cushion to ensure it is firmly attached and pushed far enough down the center of the funnel. <i>Some women may prefer to use the pump without the Let-down Massage Cushion. The Let-down Massage Cushion has petals that flex in and out to gently massage the area around the breast. This is intended to help stimulate a fast, natural milk flow. When using the breast pump without the Let-down Massage Cushion, the maximum vacuum level achievable will be higher.</i>
<i>Cracking or discoloration of pump parts</i>	◦ Avoid contact with antibacterial or abrasive detergents or cleaners as they can damage the plastic. Combinations of detergents, cleaning products, sanitizing solution, softened water and temperature fluctuations may, under certain circumstances, cause the plastic to crack. If this occurs, do not use. The pump bodies and breast milk containers are dishwasher safe but food colorings may discolor them. Follow guidelines in 'Cleaning and sanitizing your Your Pump' on page 6.
<i>Lost or broken parts</i>	◦ Contact Philips AVENT for replacements. Call 1.800.54.AVENT or visit www.philips.com/AVENT
<i>Pump not responding to pressing of the control handle</i>	◦ Ensure the pump is correctly assembled and switched on with the power indicator illuminated GREEN. ◦ Make sure the cables between the electric wall socket and the pump are correctly connected.
<i>Pump light flashing RED</i>	◦ If the pump stops working and the power light is flashing RED then a fault has been detected. ◦ 3 flashes, delay, 3 flashes, etc – the breast pump has detected a vacuum fault. Switch off the motor and check tubing and connections for obstructions or damage. ◦ 2 flashes, delay, 2 flashes, etc – the breast pump has detected that it is over the acceptable temperature. Switch off the motor. Ensure unit is not exposed to heat sources or sitting in direct sunlight. Allow unit to stabilize before trying again. ◦ Continuous flashing – the pump has detected that the control cable has been incorrectly fitted/not fitted or the cable is damaged.

Key to product base label

TYPE B APPLIED PART

GENERAL WARNING

FOLLOW OPERATING INSTRUCTIONS

ETL LISTED MARK –
CANADA/UNITED STATES

FOR INDOOR USE ONLY

CLASS II EQUIPMENT

Technical Specifications

- Mains Input Voltage 100 – 240 VAC
- Mains Input Current 1.5A
- Mains Input Frequency 50 – 60 Hz
- Supply voltage fluctuations not exceeding +/- 10% of the nominal voltage
- Ambient temperature range: +10° C to +40° C (+50° F to +104° F)
- Relative Humidity range: 30% to 75%
- Atmospheric Pressure: 700 to 1060hPa
- Equipment rated for continuous operation

FULL TWO-YEAR WARRANTY

Philips Electronics North America Corporation (USA) and Philips Electronics Ltd (CANADA) warrant each new Philips AVENT product, model SCF304, against defects in materials or workmanship for a period of two years from the date of purchase, and agrees to repair or replace any defective product without charge. **IMPORTANT:** This warranty does not cover damage resulting from accident, misuse or abuse, lack of reasonable care, or the affixing of any attachment not provided with the product. **NO RESPONSIBILITY IS ASSUMED FOR ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES.**

In order to obtain warranty service, simply call toll-free 1.800.54.AVENT.

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