




ISIS iQ UNO™

Handheld Electronic Breast Pump

Six simple steps to using your ISIS iQ UNO

1

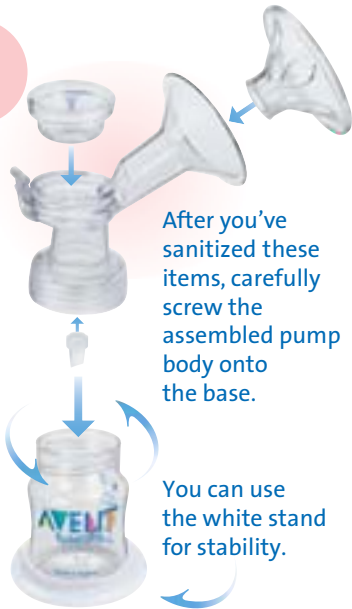

After you've sanitized these items, carefully screw the assembled pump body onto the base.

You can use the white stand for stability.

2



Push the control handle firmly onto the pump body.

3



Plug the battery pack or wall plug cord into the control handle and switch on.

4



Hold down the button for a few seconds to turn on the pump.

5



Press the pump firmly against your breast to form a seal. Use shallow, quick compressions to help stimulate let-down. Once the milk begins to flow, establish a comfortable pumping rhythm.

6



Press the blue One-Touch Memory Control, and ISIS iQ does the work for you! The pump learns and continues your gentle, effective pumping.