Foot control

LFH 2210

LFH 2310

LFH 2320

LFH 2330

For product information and support, visit
www.philips.com/dictation

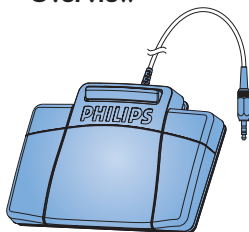
PHILIPS

EN	User manual	3
DE	Benutzerhandbuch	11
FR	Manuel de l'utilisateur	19
ES	Manual del usuario	27
NL	Gebruikershandleiding	35
IT	Manuale di istruzioni	43


1 Welcome

Congratulations on your purchase and welcome to Philips! To fully benefit from the support that Philips offers, visit our Website for support information such as user manuals, software downloads, warranty information and more: www.philips.com/dictation.


1.1 Overview

Foot control 2210
with analog connector

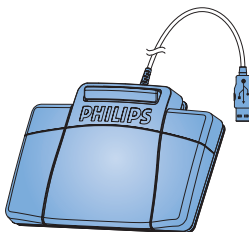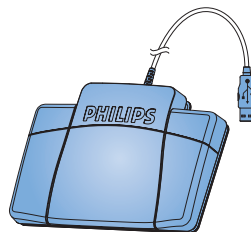Foot control 2310 / 2330
with USB connector

Foot control 2320
with USB connector

2 Important

2.1 Safety

- To avoid short circuit, do not expose the product to rain or water.
- Do not expose the device to excessive heat caused by heating equipment or direct sunlight.
- Protect the cables from being pinched, particularly at plugs and the point where they exit from the device.
- Back up your files. Philips is not responsible for any loss of data.

2.2 Disposal of your old product

- Your product is designed and manufactured with high quality materials and components, that can be recycled and re-used.
- When this crossed-out wheeled bin symbol is attached to a product, it indicates that the product is covered by the European Directive 2002/96/EC.
- Please become informed about the local separate collection system for electrical and electronic products.
- Please act according to your local rules and do not dispose of old products with your normal household waste. The correct disposal of old products will help prevent potential negative consequences for the environment and human health.

2.3 Technical support and warranty

This Philips product was designed and manufactured to the highest quality standards. If your Philips product is not working correctly or is defective, contact your supplying Philips Speech Processing dealer who will provide you with the necessary information on how to proceed.

If you have questions that your dealer cannot answer or any other related question, visit www.philips.com/dictation to contact us.

No components are user-serviceable. Do not open or remove covers and do not insert objects not suitable for the specific connectors. Repairs may be made only by Philips service centers and official repair facilities. Failure to do so will void any warranty, stated or implied. Any

operation expressly prohibited in this manual, any adjustments, or assembly procedures not recommended or authorized in this manual will void the warranty.

3 Installation

3.1 Foot control 2210 with analog connector

- 1 Connect the plug of the foot control to the foot control socket of the transcription unit.

Tip

The foot control 2210 also can be connected to a PC in combination with the optional Philips USB adapter 6220.

3.2 Foot control 2310 / 2320 / 2330 with USB connector

- 1 Connect the foot control to a free USB port on the PC.
- 2 Insert the installation CD in the CD-ROM drive. The set-up program should start automatically. If it does not, open the CD contents in Windows Explorer and double-click the [autorun.exe](#) file.
- 3 Click the [Install](#) button.
- 4 Follow the on-screen instructions of the set-up program to install the driver software on the PC.

Note

Administrator rights are required to install the software. For more help, contact your administrator.

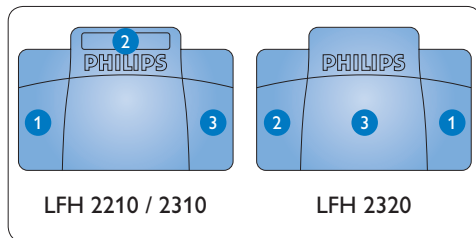
User manual

4 Use the foot control


4.1 Foot control 2210 / 2310 / 2320

The foot control has three pedals: fast rewind

1, fast forward 2 and playback 3.


- 1 Press and hold the pedal for fast rewind. When rewinding reaches the desired position, release the pedal.
- 2 Press and hold the pedal to fast forward. When forwarding reaches the desired position, release the pedal.
- 3 The pedal starts playback. The behavior for starting and stopping playback depends on the setting of the playback switch on the underside of the foot control:
 - **N (Neutral) position:** Press and hold the right pedal for playback. When the pedal is released, playback is stopped.
 - **T (Toggle) position:** Press and release the right pedal to start continuous playback. Press the right pedal again to stop playback.

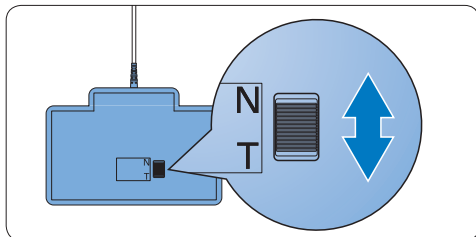4.2 Foot control 2330

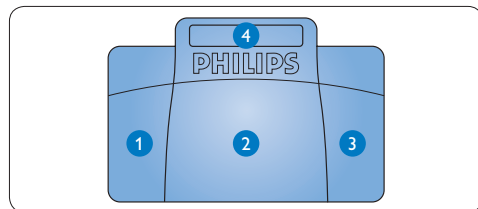The foot control has four pedals: a left **1**, a middle **2**, a right **3** and a top **4** pedal.


- 1** Press and hold the **left pedal** to fast forward. When forwarding reaches the desired position, release the pedal.
- 2** Press and release the **middle pedal** to start continuous playback. Press the middle pedal

again to stop playback.

- 3 Press and hold the **right pedal** for fast rewind. When rewinding reaches the desired position, release the pedal.
- 4 Press and release the **top pedal** to finish a dictation.

* Tip

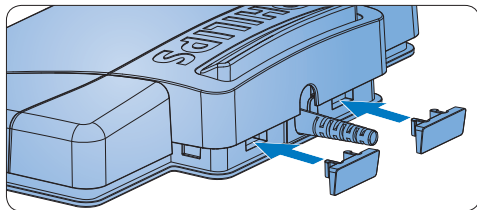If the middle pedal has been disabled with the Device Configuration Tool (see next chapter), it can be secured with the plastic locking tabs (included).

5 Advanced configuration

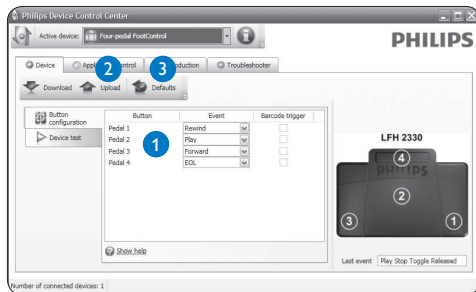
Use the [Device Control Center](#) to change the pedal functions of foot controls with USB connector.

- 1 Connect the foot control to a free USB port on the PC.
- 2 Start the Device Control Center by opening the Windows start menu and selecting [All Programs > Philips Speech Control > Philips Device Control Center](#).
- 3 Change the function of the pedals according to your preferences. 1
- 4 Click the [Upload](#) button 2 to save the new settings.

Note

The foot control can be reset to its factory default configuration by clicking the [Defaults](#) button 3. Click the [Upload](#) button 2 to save the default settings.

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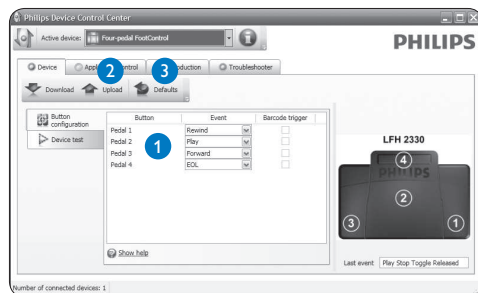
Declaration of Conformity

Tested to comply with FCC standards
FOR HOME OR OFFICE USE

Model Number: LFH 2210 / LFH 2310 / LFH 2320 / LFH 2330

Trade Name: Philips Foot Control

Responsible Party: Philips Speech Processing, 64 Perimeter Center East, Atlanta, GA 30346, USA, Tel. 888-260-6261

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- 1) This device may not cause harmful interference.
- 2) This device must accept any interference received, including interference that may cause undesired operation.

The CE mark confirms that the device corresponds to the relevant guidelines of the European Union.

The cardboard used for packaging can be recycled. This electronic device contains recyclable material.

Discover the comprehensive range of professional dictation products from Philips
www.philips.com/dictation

Specifications are subject to change without notice.

Trademarks are the property of Royal Philips Electronics or their respective owners.

© 2009 Royal Philips Electronics. All rights reserved.

Document version 4.0, 27/02/2009.

5103 109 9701.4